

# Sadler

Twin rail for curved stairs



**Stannah**

# Make a step change

## If a Stannah stairlift could make life easier, why wait?

Life is better when you have the energy to do the things you want to do, like enjoying a visit from the grandchildren or entertaining friends. Our experience has taught us that the sooner a stairlift is installed, the sooner you will be able to start enjoying the things you love doing. Just think how a Stannah stairlift could make your day-to-day life easier.

## A stairlift that's right for you and your home.

We know that not all homes are the same and that curved staircases can come in different shapes and sizes. Our twin tube rail can be the ideal choice when you need a stairlift that will follow your staircase perfectly.



Inside rail

- Our stairlifts can accommodate different types of curved staircases.
- We can install our rails on either side of the staircase as shown in the photos below.
- Our slimline rails and fold-away chairs make the best use of available space and leave enough room for others to use the staircase.
- All our rails are tailor-made, so they fit your staircase perfectly.
- Contrary to popular belief our rails are fitted to the staircase and not the wall - ensuring minimal mess and bother.
- Installation is quick and easy.
- Our stairlifts can navigate all kinds of corners and landings with a range of rail solutions.



Outside rail

# A company you can trust

**We are a British, family-run and owned company and have been for five generations.**

The current managing directors are the great, great grandchildren of Joseph Stannah, who started the company in the 1860s.

His engineering principles and strong family values run as true today as they always have.

Now, over 150 years later, we have manufactured more than 700,000 stairlifts and installed products in over 40 countries.

## **Sadler: Stylish, sleek and very comfortable**

Designed specifically for people with limited flexibility, the Sadler is a totally new concept in stairlift design. It offers a new alternative to traditional and perch-style stairlifts, helping you maintain a more upright posture at the same time as taking much of your weight through the seat, minimising the weight supported by your legs.

The slim design is also a great alternative for homes where the staircase is too narrow to accommodate a traditional stairlift.

The elegant Sadler is crafted for choice and ease-of-use in every way, with the chair folding away for a compact footprint allowing other people to continue to use the stairs. It's this level of thought and care that ensures that Sadler is the natural choice.



The Sadler's slimline design allows others to still use the stairs

# Sadler key features

## 1. Arm-to-footrest link

When not using your stairlift, you can fold away the arms and the footrest will fold automatically. There's no awkward bending down, which is particularly useful at the top of the stairs.

## 2. Saddle seat

The unique saddle style seat enables you to maintain a more upright posture and minimises the weight supported by your legs. The assisted lift feature offers easy entry and exit from the chair.

## 3. Seatbelt

The easy-to-use seatbelt fastens and unfastens with one hand and it's easy on arthritic hands. The stairlift will not operate unless the seatbelt is securely fastened.

## 4. Easy-to-use controls

Ride the Sadler using our simple, intuitive controls that have been designed for those with less dexterity. They work with the lightest of touches, and can be operated using your hand, arm or wrist.



## 5. Safety edges

If something obstructs the stairlift, sensors on the edge of the carriage and footrest will automatically bring the stairlift to a stop.

## 6. Wall controls



Our clever wall controls are ideal for when you need to summon your stairlift from another floor, or perhaps send it up or down to someone else who needs to use your Sadler.

## Sadler key benefits

**Effortless and simple. This sums up beautifully just how much thought and care we have designed into each and every function of our Sadler stairlift. From the safe and easy-to-use seatbelts, to stairlifts that fit on either side of the stairs and on all types of staircases, we have made your Sadler stairlift simple, safe and reliable.**



Seatbelt can be fastened with one hand



Sensors detect obstructions on your stairs



Fold the arms and the footrest folds automatically

**“We are delighted with the stairlift. My wife enjoys the ride. It gives her independence. She finds it very easy to use”**

**Mr A, Cirencester**



## The easy way to climb your stairs

**Everything about a Stannah stairlift is made to make managing your stairs as easy, comfortable and safe as it can be, so your life changes for the better the moment you begin to use it.**

**From the simple controls at your fingertips to reliable operation and a flexible and compact design, we have done all the hard work so you don't have to. Just settle into the comfortable seat and sit back, as you travel up or down your stairs with ease.**

We really have taken care of every detail to ensure your Sadler stairlift is as effortless to use as it is easy to operate.

**Step 1:** Simply unfold the arms of your Sadler and the footrest will safely unfold into position.

**Step 2:** Once unfolded, sit on your stairlift and make yourself comfortable in the seat. Then pull across and fasten

the easy reach seatbelt and you're now ready to go.

**Step 3:** Gently apply pressure to the easy-to-use light touch controls in the direction you wish to travel and your Sadler will glide smoothly up or down the stairs.

**Step 4:** When you arrive at the top of the stairs, the rail will continue onto your

landing so that you can alight safely and securely from your Sadler.

**Step 5:** When you are not using your Sadler, the arms and footrest fold neatly away, allowing more space for other stair users to pass safely.

**“Now we have our Stannah stairlift, we no longer have to face an unwanted move into single storey accommodation”**

**Mr & Mrs H, Worcestershire**



## Tailored to you

Your comfort and safety mean everything to Stannah. This is why we have designed the Sadler for complete peace-of-mind, giving it a range of helpful features that make it safe, easy and enjoyable to use.





### Control choices and control position



Light-touch control

Left hand     Right hand



Rudder style control

### Illuminated power button



### Wall controls



### Tilting seat



### Seatbelt






# Sadler colour choices

## Woven upholstery



Stone Woven

## Rail colour options

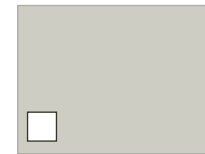

Pearl (Standard)



Nut Brown



Jet Black



Window Grey



Beige



Ochre Brown

The pictures within our brochures are for guidance only. Although we have tried to display the colours accurately, we cannot guarantee that the colours in the pictures accurately reflect the colour of the actual stairlifts. Your stairlift may be slightly different from these pictures.

## A professional installation

When you have chosen the Sadler that suits you best, a Stannah trained engineer will visit your home and professionally install your new stairlift in a matter of hours, leaving your home just as they found it.





Our company policy is one of continuing product improvement. We therefore reserve the right to alter the specifications and to change and improve design at any time. Full operating instructions are supplied for all products. Stannah is a registered trademark. The CE mark indicates that this product complies with the applicable European directives which relate to health, safety, environment and consumer protection. Design, manufacture and installation: Stannah Stairlifts Ltd has been accredited to ISO9001:2008 since 2009, this is for design, development, production and distribution of stair lifts. Alterations in models, standard and optional equipment as described in the text and illustrations, may occur. For precise information please contact Stannah.

© Stannah Stairlifts Ltd 2020. Not to be reproduced wholly or in part without the written permission of Stannah Stairlifts Ltd Issue 1 / January 2020

**Dolphin Stairlifts  
(East Anglia Ltd)**  
25 Estuary Drive,  
Felixstowe, Suffolk, IP11 9TL

01394 277100  
office@dolphinliftsanglia.com

**Stannah**

AUTHORISED DEALER